



Quinn's Advantage® Fertilization (HTF) Medium

For laboratory procedures only; other uses
must be qualified by the end user.

Product Description	REF Number	Unit Size
Quinn's Advantage® Fertilization (HTF) Medium	ART-1020	50 mL
Quinn's Advantage® Fertilization (HTF) Medium	ART-1021	100 mL

INTENDED USE

Quinn's Advantage® Fertilization Medium was developed for in vitro procedures involving the fertilization of human oocytes. This medium is a modification of Human Tubal Fluid (HTF) that was first described by Quinn and colleagues (*Fertil Steril*. 1984;41:202, 1985;44:493). Modifications to the original formulation include low phosphate and the addition of citrate, EDTA, selected nonessential amino acids and taurine, alanyl-glutamine as a stable source of glutamine, and the presence of lactate in the form of calcium lactate.

COMPONENTS

Sodium chloride
Potassium chloride
Magnesium sulfate
Potassium phosphate
Calcium lactate
Sodium bicarbonate
Glucose
Sodium pyruvate
Alanyl-glutamine
Taurine
L-Asparagine
L-Aspartic acid
Glycine
L-Proline
L-Serine
Sodium citrate
EDTA
Gentamicin
Phenol red

STORAGE INSTRUCTIONS AND STABILITY

Store unopened containers refrigerated at 2°C to 8°C. Warm to incubator (37°C) temperature and equilibrate with desired atmosphere containing 5% CO₂ prior to use. Do not freeze or expose to temperatures greater than 39°C. It is advisable to regass the medium with 5% CO₂ before replacing opened bottles back in the refrigerator.

In order to ensure optimal performance, SAGE Media™ strongly recommends measuring the pH of the

medium under laboratory working conditions and adjusting the level of CO₂ used to attain the desired pH range for optimal embryo development. The desired pH range for Quinn's Advantage® Fertilization (HTF) Medium is 7.3 ± 0.1. The product is stable until the expiration date shown on the label or within 30 days of the Date of First Use provided that proper aseptic procedures have been observed by the user:

- Remove desired volume of product using aseptic procedures.
- Once product has been removed from the original container, regass the container and reseal it to ensure a tight seal. Write the date the product was first opened on the product label. Do not use product longer than 30 days after opening the container.
- Once removed, do not return any volume of product to the original container.
- Once the product has been opened, store the sealed container at 2°C to 8°C.
- Do not use if the product becomes discolored, cloudy, turbid, or shows any evidence of microbial contamination.

One-cell MEA tested and passed with 80% or greater blastocyst. USP Endotoxin gel clot tested and passed with <1 EU/mL.

A Certificate of Analysis is available for this product.

DIRECTIONS FOR USE

This is the preferred medium for fertilization procedures in vitro. We currently recommend the use of Quinn's Advantage® Cleavage Medium (REF #1026/1027) for culture of embryos from Day 1 until Day 3, and Quinn's Advantage® Blastocyst Medium (REF #1029) for culture from Day 3 to Day 5/6. The media should be supplemented with an exogenous protein source such as Human Serum Albumin (REF #3001/3003) or Serum Protein Substitute (REF #3010/3011).

Each laboratory should make its own determination of which medium to use for each particular procedure.

Information on specific aspects of IVF, embryo culture, and cryopreservation is available in our Product Catalog (REF #80572).

PRECAUTIONS AND WARNINGS

Do not use medium that shows evidence of particulate matter, cloudiness, or is not rose colored.

To avoid problems with contamination, practice aseptic technique and discard minimal amounts of excess medium remaining in the bottle.

Caution: Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

RELATED PRODUCTS

SAGE Assisted Reproduction Products™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE Support Line at the number below.

Quinn's Advantage® is a registered trademark of CooperSurgical, Inc.

Patrick Quinn, PhD, HCLD, SAGE IVF, Inc.

Call the **SAGE SUPPORT LINE** at:
1-800-243-2974 or 1-203-601-5200



SAGE In-Vitro Fertilization, Inc. A CooperSurgical Company
95 Corporate Drive, Trumbull, CT 06611
www.coopersurgical.com Part #SPN-00432 • Rev. 3/03