



Sterile Oil for Tissue Culture

For laboratory procedures only; other uses must be qualified by the end user.

Product Description	REF Number	Unit Size
Oil for Tissue Culture	ART-4008P	100 mL
Oil for Tissue Culture	ART-4008-5P	500 mL

INTENDED USE

Oil for Tissue Culture is intended for use in laboratory procedures that may require small volumes of tissue culture medium to be covered with a relatively inert, nonmiscible liquid component that allows diffusion of gases. Such procedures include in vitro fertilization and embryo culture and micromanipulative procedures such as ICSI and assisted hatching. Some of the proposed benefits of using small volumes of medium under oil include minimal evaporation of water from the medium, culture of gametes and embryos at increased cell densities, and the use of smaller volumes of medium containing expensive components.

COMPONENTS

Liquid petrolatum/paraffin oil/low-viscosity mineral oil. Sterile filtered. (SAL 10⁻³)

STORAGE INSTRUCTIONS AND STABILITY

Store unopened containers at room temperature. The product is stable until the expiration date shown on the label or within 30 days of the Date of First Use provided that proper aseptic procedures have been observed by the user.

- Remove desired volume of oil from the bottle using a sterile non-toxic pipette. DO NOT POUR.
- Once product has been removed from the original container, reseal the container to ensure a tight seal. Write the date the product was first opened on the product label. Do not use product longer than 30 days after opening the container.
- Once removed, do not return any volume of product to the original container.
- Once the product has been opened, store the sealed container at room temperature.
- Do not use if the product becomes discolored, cloudy, turbid, or shows any evidence of microbial contamination.

One-cell MEA tested and passed with 80% or greater blastocyst.

A Certificate of Analysis is available for this product.

DIRECTIONS FOR USE

It has been recommended to equilibrate the oil with 10% (v/v) of culture medium, replacing the first lot of medium with a second lot after overnight incubation in a CO₂ incubator. The container of equilibrated oil is then left in the incubator with the cap of the bottle loosely in place to allow gas equilibration.

We have found that nonequilibrated oil, used directly from the bottle as delivered, gives results in human ART and mouse embryo culture equivalent to those obtained with equilibrated oil.

Each laboratory should make its own determination of which medium to use for each particular procedure.

Information on specific aspects of IVF, embryo culture, and cryopreservation is available in our Product Catalog (REF #80572).

PRECAUTIONS AND WARNINGS

Fat-soluble compounds such as steroids and fat-soluble vitamins will preferentially diffuse into the oil layer from aqueous culture medium. Spilt product can be removed with 70% alcohol and paper towel.

Do not use any container of oil that shows evidence of particulate matter or microbial contamination.

To avoid problems with contamination, practice aseptic technique and discard minimal amounts of excess product remaining in the bottle.


Caution: Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

RELATED PRODUCTS

SAGE Assisted Reproduction Products™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE Support Line at the number below.

Patrick Quinn, PhD, HCLD, SAGE IVF, Inc.

Call the **SAGE SUPPORT LINE** at:
1-800-243-2974 or **1-203-601-5200**

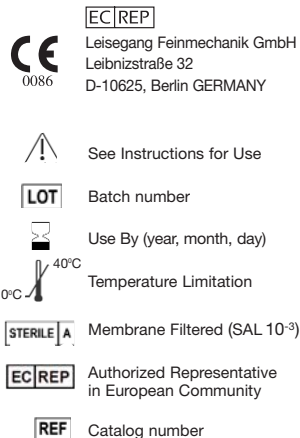

SAGE In-Vitro Fertilization, Inc. A CooperSurgical Company
95 Corporate Drive, Trumbull, CT 06611
www.coopersurgical.com